



Tetric EvoFlow®

1

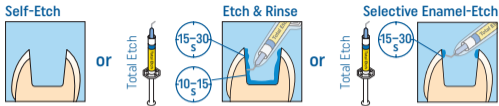


2

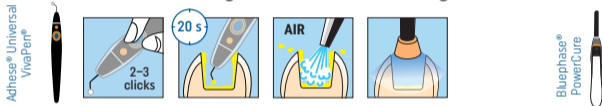

3



4

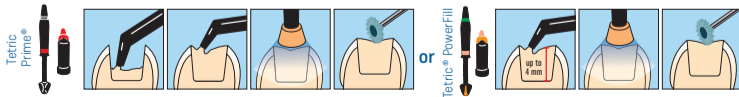

mW/cm ²	Tetric EvoFlow/ Tetric Prime	Tetric PowerFlow/ Tetric PowerFill	Adhese Universal
500 – 900	20 s	20 s	10 s
1,000 – 1,300	10 s	10 s	10 s
1,800 – 2,200	5 s	5 s	5 s
2,700 – 3,300	–	3 s	3 s

For permanent teeth only
For deciduous teeth see instructions for use

CE 0123

Rx ONLY
For dental use only!

see
Instructions
www.ivoclar.com/eIFU

Date information
prepared: 2023-07-10
759556/Rev. 1


ivoclar

Tetric® Line

4-mm composites

SCULPTABLE
Tetric® PowerFill

FLOWABLE
Tetric® PowerFlow

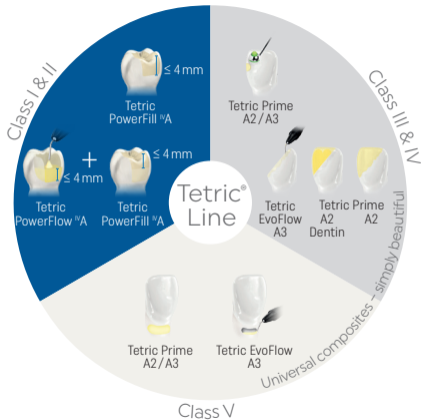

Conventional composites

SCULPTABLE
Tetric® Prime

FLOWABLE
Tetric EvoFlow®