ProTaper Gold®

EN	Instructions For Use	2
FR	Instructions d'utilisation	7
IT	Istruzioni per l'uso	13
DE	Gebrauchsanweisung	19
ES	Instrucciones de uso	25
PT	Instruções de Utilização	31
NL	Gebruiksaanwijzing	37
SV	Användarinstruktioner	43
DA	Brugsanvisning	48
NO	Brukerinstruksjoner	53
LT	Naudojimo instrukcija	58
LV	Lietošanas norādījumi	63
ET	Kasutusjuhised	68
CS	Návod k použití	73
SK	Návod na použitie	78
HU	Használati utasítás	83
RO	Instrucțiuni de utilizare	89
SR	Uputstva za upotrebu	95
BS	Uputstva za upotrebu	100
CNR	Uputstva za upotrebu	105
SQ	Udhëzimet për përdorim	110
EL	Οδηγίες χρήσης	116
TR	Kullanım Kılavuzu	122
PT-BR	Instruções de uso	127
KO	사용 지침	133

ProTaper Gold®

EN

*FOR DENTAL USE ONLY
FOR PROFESSIONAL USE ONLY
STERILE – STERILIZED BY RADIATION
LIABLE FOR REPROCESSING*

INSTRUCTIONS FOR USE REUSABLE STERILE ENDODONTIC INSTRUMENTS

0) INDICATIONS FOR USE

- The products are indicated for treatment of endodontic disease.
- Intended purpose: Engine-driven instrument intended for root canal preparation (shaping and debridement of the root canal).
- Endodontic instruments are to be used only in a clinical or hospital environment, following good dental practice, by qualified dental professionals such as general practitioners as well as Endo specialists (Endodontist) and Dental Assistants.
- Instruments shall be used in combination with an endodontic motor (manually in severe curvatures).

1) CONTRAINDICATIONS

None.

2) CONTENT, COMPOSITION AND COMPATIBLE DEVICES

ProTaper Gold® instruments are made of three main components: the working part made of a nickel-titanium alloy, a colored silicone stopper and a plated brass shank with a colored ring (except for instrument SX).

The files are manufactured with a process that results in a file that has a colored appearance.

Due to this proprietary processing, the files may appear slightly curved. This is not a manufacturing defect. While the file can be easily straightened using only your fingers, it is not necessary to straighten the file prior to use. Once inside the canal, the file will follow the root canal anatomy.

The ProTaper Gold® range comprises following endodontic instruments:

Code xx-length y-file size	Product	ISO XXX YYYv XXX-ISO size YYY-taper size v-variable taper	Available lengths	Packaging and content
A 0410 2xx G0y 03	ProTaper Gold® Shaping files	SX: ISO 019 004v S1: ISO 018 002v S2: ISO 020 004v	19 mm 21/25/31 mm	Blister with 6 instruments (sterile)
A 0411 2xx G0y 03	ProTaper Gold® Finishing files	F1: ISO 020 007v F2: ISO 025 008v F3: ISO 030 009v F4: ISO 040 006v F5: ISO 050 005v	21/25/31 mm	Blister with 6 instruments (sterile)
A 0409 2xx G01 03	ProTaper Gold® Assortment	SX S1 S2 F1 F2 F3	19 mm 21/25/31 mm	Blister with 6 instruments (sterile)
A 0409 2xx GD1 03	ProTaper Gold® DEMO PACK/ Assortment	SX S1 S2 F1 F2 F3	19 mm 21/25/31 mm	Blister with 6 instruments (sterile)

- Torque control devices ensure optimal usage.
- Use with endodontic motors in a constant rotation at a speed of 250 - 350 rpm.
- Set torque at:

File Size	Torque [Ncm]	Speed [rpm]
ProTaper Gold® S1 ProTaper Gold® SX	5-6	300
ProTaper Gold® S2 ProTaper Gold® F1	1.5-2	
ProTaper Gold® F2 ProTaper Gold® F3 ProTaper Gold® F4 ProTaper Gold® F5	3-4	

- Lubricants such as NaOCl, EDTA, ProLube, Glyde™ shall be used.
- The use of radiographs in combination with an apex locator and a tool for adjusting the silicon stopper to the correct working length is the appropriate method of working length determination.

3) WARNINGS

- Strictly follow this Instruction for Use and the General Processing Instructions For Endodontic Products (see section 8) to minimize following risks to the device, the patient and/or the user:
 - Breakage of instrument.

- Cross-contamination.
- Heat generation due to insufficient lubrication and irrigation.
- Swallowing of working part of the instrument.
- Toxic or allergic reactions caused by processing residues.
- Similar to all machine-driven root canal instruments, ProTaper Gold® instruments should not be used in a root canal with an abrupt apical curvature due to heightened risk of separation. In this case, pre-curved hand files should be used in the apical region.

4) PRECAUTIONS

- Safety and effectiveness of use have not been established in pregnant or breastfeeding women or in children.
- Use a rubber dam system during the endodontic procedure.
- For your own safety, wear personal protective equipment (gloves, glasses, mask).
- Inspect the packaging before use and do not use the instruments if the packaging is damaged.
- Do not use the instruments after expiration date.
- Check the instrument before each use for signs of defects such as deformations (bent, unwound), breakage, corrosion, damaged cutting edges, loss of color coding or marking. With these indications the devices are not able to fulfil the intended use with the required safety level, instruments should be discarded.
- Before using any instrument, make sure it is well connected to the contra-angle head.
- Check instrument and clean working part frequently during instrumentation, inspecting for signs of distortion, elongation or wear, such as uneven flutes, dull spots. With these indications that the devices are not able to fulfil the intended use with the required safety level, instruments should be discarded.
- Instruments should not be fully immersed in a sodium hypochlorite solution (NaOCl). Only the working part of the NiTi instrument, which is in contact with the patient should be immersed in a NaOCl solution concentrate at NOT more than 5%.
- Exercise caution in the apical area and in canals that divide, and/or exhibit abrupt curvatures or recurvatures.
- Irrigate abundantly and frequently the canal throughout the procedure and after each instrument used (according to good dental practices).
- Always use minimal apical pressure. Never force the files down the canal.
- When instrument does not easily progress, clean and inspect the cutting flutes, then irrigate, recapitulate with a manual file and reirrigate.
- For shaping extremely curved canals it is safer to use the file only to shape one canal in order to reduce the risk of breakage. Pay attention to the following good practices:
 - Use a new file and discard it after the canal was treated (single canal use).
 - Use manual instead of rotary files.
 - Use small size, flexible or/and NiTi files (this will help avoid canal transportation).

- Visually inspect the working part for all the defects listed in the former paragraph during use (i.e after each wave).
- Avoid the standard reaming continual rotational motion and instead use small angle motions (filing motion, watch winding oscillation motion, or balanced force technique) in order to limit the rotational bending fatigue on the instruments and improve their expected life.

5) ADVERSE REACTIONS

In the present technical state, no adverse reaction has been reported so far.

6) STORAGE CONDITIONS

Keep products in a dry and clean environment, away from sources of moisture and direct sunlight.

7) STEP BY STEP INSTRUCTIONS

- 1) Review different horizontally angulated radiographs to diagnostically determine the width, length, and curvature of any given root canal.
- 2) Create straight-line access to the canal orifice(s) with emphasis on flaring, flattening, and finishing the internal axial walls.
- 3) Irrigate abundantly and frequently after each instrument used (according to good dental practices).
- 4) The ProTaper Gold® sequence is the same regardless of the length, diameter or curvature of the canal.
- 5) Passively scout the coronal 2/3 with hand files ISO 010 and ISO 015, in the presence of a viscous chelator (such as Glyde File Prep™ root canal conditioner).
- 6) Gently work these instruments until a smooth, reproducible glide path is confirmed. Alternatively, mechanized glide path files (such as ProGlider® or PathFile®) may be used after an ISO 010 hand file.
- 7) In the presence of NaOCl, brush and follow along the glide path with Shaping File S1. Before light resistance is encountered, laterally brush and cut dentin on the outstroke to improve straight-line access and apical progression. Always brush away from the furcation.
- 8) Use passively an ISO 015 hand file to resistance.
- 9) Use Shaping File S1 with brushing action until the same depth as the ISO 015 hand file is reached.
- 10) Repeat this sequence with Shaping File S2, exactly as described for the Shaping File S1, until the depth of the ISO 015 hand file is reached.
- 11) In the presence of a viscous chelator or NaOCl, scout the apical 1/3 with ISO 010 and 015 hand files and gently work them until they are loose at length.
- 12) Establish working length, confirm patency and verify the presence of a smooth reproducible glide path in the apical 1/3.
- 13) Use first Shaping File S1 and then Shaping File S2, with a brushing action, until working length is reached.
- 14) Reconfirm working length, irrigate, recapitulate and re-irrigate, especially in more curved canals.

- 15) Use Finishing File F1, in a non-brushing action, with each insertion deeper than the previous insertion until working length is reached. Do not leave the file at working length for longer than one second.
- 16) Gauge the foramen with an ISO 020 hand file:
 - If the instrument is snug at length, preparation is finished and the canal is ready to be obturated.
 - If the instrument is loose at length, continue with Finishing File F2 and, when necessary proceed with Finishing Files F3, F4 and F5, with the same non-brushing motion to working length, gauging after each Finishing File respectively with a hand files of the same size (ISO 025, ISO 030, ISO 040 or ISO 050).
- 17) If necessary, use Shaping File SX with a brushing action to move the coronal aspect of the canal away from furcation and/ or to create more coronal shape. SX can also be used to optimally shape canals in shorter roots.
- 18) Use dedicated paper points to dry and dedicated gutta-percha points to obturate the root canals.

8) HYGIENE, DISINFECTION, CLEANING AND STERILIZATION

- Please follow the General Processing Instructions For Endodontic Products (<https://www.dentsplysirona.com/en/service-contact/ifu.html>) for the disinfection, cleaning and sterilization steps.
- Do not re-use silicone stops. Remove and discard silicone stops after each use.
- Products shall be disposed according to local regulations for the safe disposal of sharp and contaminated devices.

9) ADDITIONAL INFORMATION

- Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
- Sterility cannot be guaranteed if packaging is open, damaged or wet.
- To get a free printed copy of IFU please see section “Get a print copy of IFU” on website <https://www.dentsplysirona.com/en/service-contact/ifu.html>.
- Explanation of non-harmonized symbols for IFUs and labels, see IFU Symbols (<https://www.dentsplysirona.com/en/service-contact/ifu/ifu-symbols.html>).

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DENTSPLY DeTrey GmbH